

SLEEP PARTNER


眠尔安



*Life is so Beautiful, Love is Always
With Your Sweet Dream.*

This clinically proven safe and effective bracelet now allows you to awake refreshed from a restful night's sleep without any drug hangover. It is also very effective for day sleepers. There is a health part in the bracelet. It is specially designed for improving insomnia. When you need treatment of insomnia, please take the little sticker on the back off, and put the holes directly on the Shenmen acupuncture point. The bracelet allows electrical impulses to influence the acupuncture point allowing you to obtain the sleep which is so important to your overall health. For the treatment of insomnia, it should be worn 30 minutes to one hour before sleep and taken off the next morning.

USE METHOD:

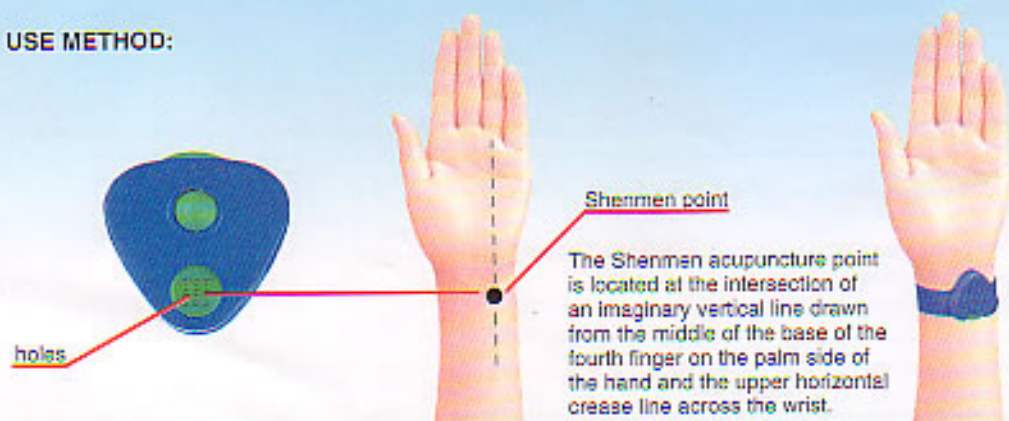

Shenmen point

The Shenmen acupuncture point is located at the intersection of an imaginary vertical line drawn from the middle of the base of the fourth finger on the palm side of the hand and the upper horizontal crease line across the wrist.

holes

aim the holes at the Shenmen points

SLEEP PARTNER


眠尔安



The bracelet also has another function:

There is a small ball on the back of the bracelet. When you feel dizzy or vomiting while taking cars, ships, or airplanes, put the small ball on the Neiguan acupuncture point and stimulate the point by adjusting pressure with the gradual turning of the knob this will be helpful to improve these diseases.

USE METHOD :

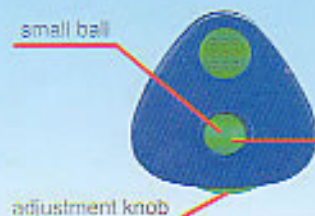

The Neiguan acupuncture point is located at the intersection of an imaginary vertical line drawn from the middle of the base of the middle finger on the palm side of the hand and a horizontal line drawn two fingers width (the index and middle finger) above the lower crease line across the wrist.



aim the small ball at the Neiguan point

CAUTION:

The bracelet can be worn on left or right wrist, or on both wrists at the same time if necessary.
Avoid moisture when wearing and store it in dry place.
The bracelet is factory sealed and should not be opened by consumers.

TERM OF VALIDITY: one year

East West Development, Inc.
Headquarters
4920 SOM Center Road
Moreland Hills, Ohio 44022
Tel: (440) 349-2064
Fax: (440) 349-1769
Email: sales@qualitylifeintl.com